

PACIFIER SYRINGE SET



**BPA
FREE**

5PC SET!

1 SYRINGE, 1 PACIFIER, 1 MEASURING CUP, 1 TUBE BRUSH, 1 STORAGE CASE

Simple Steps

1

Fill the measuring cup with the accurate dose of liquid.

2

Insert the filled syringe into the pacifier medicine tube with slight twist.

3

Place pacifier in baby's mouth and press the syringe slowly to release the fluid.

4

Twist syringe out and follow cleaning instructions after each use.



Cleaning Instructions

Fill syringe with warm soapy water, attach pacifier and plunge to clean. Use the tube brush to get to the hard-to-reach spaces. Rinse and air.

⚠ WARNING: This product is not a toy. Use only under adult supervision. Always follow prescribed dosage. Do not microwave or steam. Inspect nipple regularly for tear, crack or signs of wear. Dispose of nipple if defects are found.
CHOKING HAZARD - Small parts. Not for children under 3 yrs.